the Spa Seat™

SWIVEL 360 Shower Seat

Proper use:

Only use the stool in accordance with the instructions.

Do not use as a step, as training equipment, or as a climbing aid.

The stool is suitable for a maximum body weight of 300 lbs (136 kg).

Please do not use the stool if you weigh more than 136 kg.

Permanent damage may be caused to the stool if the maximum weight is exceeded.

Do not remove legs once assembled.

This product is intended for adults and not to be used by babies or small children, whether supervised or unsupervised.

Cleaning:

Clean from time to time with a cloth.

You may use stain-removing cleaning agents.

Do not use cleaning agents containing alkali, acid, chlorine and abrasive additives.

Liability exclusion:

Better Living is not responsible for injuries and damage arising from the non-observance of the instructions and safety measures stated above.

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Better Living Products International, Inc.

201 Chrislea Road
Vaughan, Ontario
L4L 8N6
Canada

Phone: (905) 264-7100
Fax: (905) 264-3690
Toll Free: 1 (800) 487-3300

Téléphone: (905) 264-7100
Télécopieur: (905) 264-3690
Sans frais: 1 (800) 487-3300

Teléfono: (905) 264-7100
Fax: (905) 264-3690
Sin cargo: 1 (800) 487-3300

If outside North America, your local distributor contact information is available at:

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BETTER LIVING™
making life a little easier

**PLEASE DO NOT RETURN THIS PRODUCT TO THE STORE
IL NE FAUT PAS RAPPORTER CE PRODUIT AU MAGASIN
POR FAVOR, NO DEVUELVA ESTE PRODUCTO A LA TIENDA**

ATTENTION!

Thank you for purchasing a product from Better Living. Our products are manufactured to the highest standard of quality and we stand behind them 100%. Many times, an issue can be resolved over the phone. If you are experiencing a technical issue, have questions regarding the operation of your product, or are in need of warranty assistance, please contact our Customer Support line at

1.800.487.3300

Monday through Friday 8:30 am to 5:00 pm (EST)

If outside North America, your local distributor contact information is available at:

www.betterlivingproducts.ca/distributors

If necessary, we may repair or replace this product.

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